**Quick Start Guide** **EN**

**Schnelleinstieg** **DE**

**Guida rapida** **IT**

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# Quick Start Guide

## UNEEG™ EpiSight Analyzer

Version number 2.3

Manufacturer:  
AIT - Austrian Institute of Technology GmbH;  
Giefinggasse 4; 1210 Vienna; Austria  
Date of manufacture: 2024

Use-by date:  
Windows 10: 10/2025  
Windows Server 2019: 10/2025  
Windows 11: 11/2025

Notified Body:  
mdc medical device certification GmbH;  
Kriegerstrasse 6; 70191 Stuttgart; Germany

Medical device

Caution: Please read paragraphs marked with this symbol carefully as they contain important safety information.

(01)09120109830276

Issuing date of this document:  
2024-09

# Table of Content

Table of Content.....	3
Warnings and precautions.....	4
Contraindications.....	5
Possible adverse effects.....	5
Quick Start Guide .....	6
Installation and license activation .....	7
Patient Database .....	8
Trending .....	9
Spectrogram.....	10
Report .....	11
Keyboard shortcuts.....	12

# Warnings and precautions

UNEEG™ EpiSight Analyzer is CE-certified as a medical device. If your country requires any other approval for medical devices you must not use UNEEG™ EpiSight Analyzer.

The modules from UNEEG™ EpiSight Analyzer for the automatic analysis of EEG cannot replace the examination by the physician. As for any other automatic procedure, there can be inaccuracies during the analysis with UNEEG™ EpiSight Analyzer. The original EEG still needs to be used for the evaluation and the results of UNEEG™ EpiSight Analyzer need to be confirmed based on the unaltered raw EEG trace.

UNEEG™ EpiSight Analyzer does not provide any diagnosis or diagnostic recommendation, conclusion or prediction of the patient's state. It remains the responsibility of the physician to decide on the diagnosis or to induce treatment.

The use of the software cannot replace the real-time surveillance by medical staff particularly of the vital functions. The patient's safety remains the responsibility of the medical professional.

The maximum length for continuous EEG recording that can be reviewed and analyzed with the EEG viewer is 15 months. If a recording is longer than 15 months UNEEG™ EpiSight Analyzer will stop processing.

For UNEEG™ EpiSight Analyzer seizure detection, detection of rhythmic pattern, aEEG, background frequency and sleep analysis the EEG of at least three subcutaneous electrodes must be available.

UNEEG™ EpiSight Analyzer seizure detection cannot be used as an alarm system, since there may be a delay of several minutes between the clinical onset of a seizure and the display of the detected pattern. Some EEG-events with epileptic seizures that have an electrographic correlate might be missed. For this reason UNEEG™ EpiSight Analyzer can only support but can not replace the qualified expert in the real-time monitoring.

UNEEG™ EpiSight Analyzer detection of rhythmic pattern, aEEG, background frequency or sleep analysis included in UNEEG™ EpiSight Analyzer should not be used as an alarm system, since the visualization of patterns could be delayed by up to several minutes.

Some events, where patterns defined in the Standardized Critical Care EEG Terminology of the American Clinical Neurophysiology Society are visible in the EEG, might be missed. All measures of UNEEG™ EpiSight Analyzer must be evaluated in combination with the original raw EEG and must not be the only basis for a diagnosis, treatment or change of treatment.

# Contraindications

There are no known contraindications for the use of UNEED™ EpiSight Analyzer.

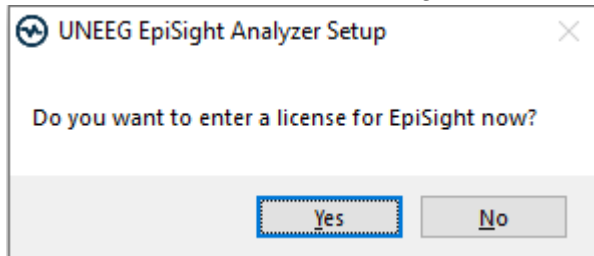
# Possible adverse effects

There are no known possible adverse effects when using UNEED™ EpiSight Analyzer.

# Quick Start Guide


## Installation and license activation

1. Download the installer for UNEEG™ EpiSight Analyzer.
2. Start the installer by double-clicking the downloaded executable.
3. The installer will guide you through the installation process.
4. After the installation the following window will pop up:


If you click “Yes” you will proceed at step 9.

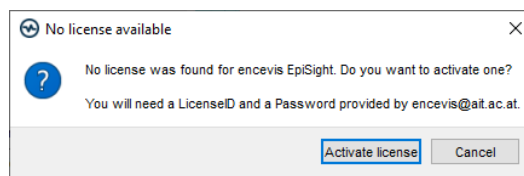5. Once the installation has finished, start the EpiSight Analyzer by clicking on the icon on your desktop.
6. You can add new patients to the database and import data.
7. The following steps are just necessary if you have no valid License activated so far. In order to open the patient’s EEG, you will need an active license.
8. Click on “*Open patient*”, the following window will pop up:

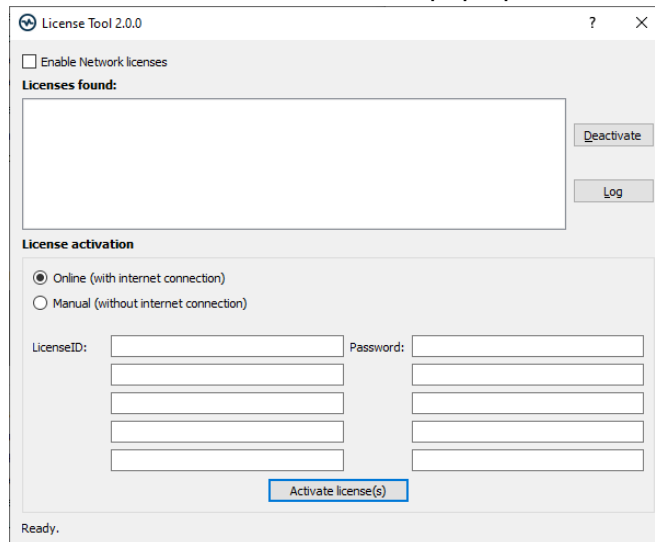9. Press “*Activate license*”, the License Tool will pop up.

10. If you are connected to the internet, select „*Online*“ as activation mode. Otherwise, select “*Manual*“ and contact your distributor in order to get the password for the manual activation.

11. Enter your LicenseID and Password.

12. Press “*Activate license(s)*“.

## Patient Database

Starting UNEEG™ EpiSight Analyzer will open the patient database window.

The screenshot shows the UNEEG EpiSight Analyzer interface. The top navigation bar includes 'Search Patient', 'Clear search', 'Select columns', 'Add new patient', 'Check connection', 'Change settings', and 'Open Help'. The main area is divided into a 'Patients' list on the left and a 'Details for Michael Jones - Connected' panel on the right. The 'Patients' list has columns for Patient ID, Last name, First name, Birth date, Connection, and Open. The 'Details' panel shows patient information, a list of UNEEG devices, a comment field, and data information including recording times and a data overview chart. Arrows point from text labels to these specific UI elements.

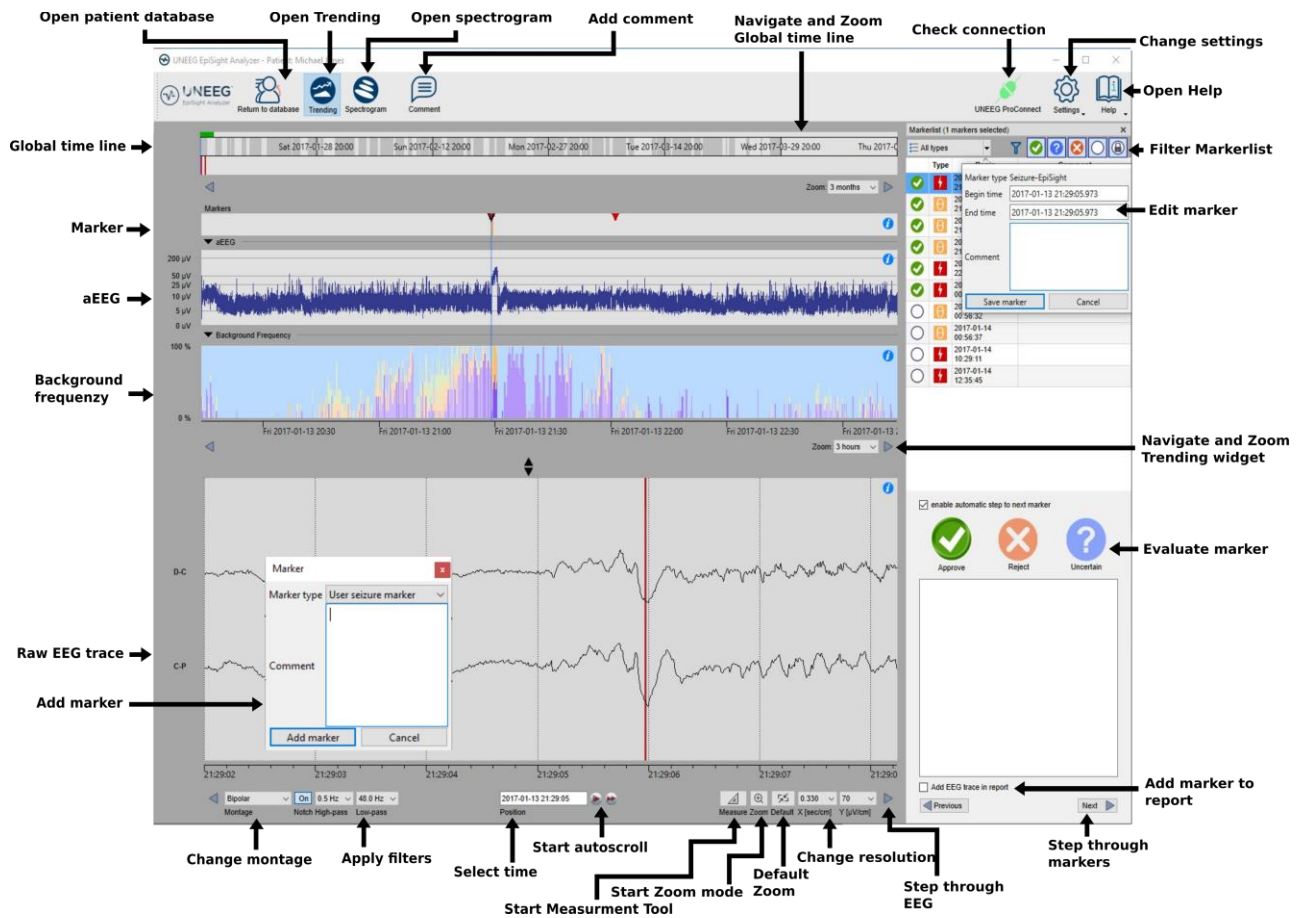
The patient database offers the following functionalities:

- Search for a patient by ID, name or date of birth
- Select the columns shown in the patient list
- Add a new patient
- Edit the patient's information
- Get the information about the EEG data available for each patient
- Open the patient's EEG
- Generate a report
- Connect to UNEEG™ ProConnect
- Import data by connecting to UNEEG™ ProConnect
- Import data manually by selecting a folder
- Change settings
- Open help

## Trending

Clicking on “Open patient” will open the trending window.

The trending includes the following functions and traces:

- Global timeline with an overview over all available data
- Zoom in the global timeline and in the analysis windows over a drop-down menu or the mouse wheel
- Seizure and other markers in a trace and in a list on the right side of the window
- Rhythmic pattern markers
- Amplitude-integrated EEG (aEEG)
- Background frequency
- EEG plot synchronized to the traces above
- Change montage
- Apply notch, high-pass and low-pass filters
- Change resolution in time and voltage
- Apply standard zoom
- Start zoom mode
- Start measurement Tool
- Start autoscroll
- Add markers
- Review and edit markers
- Filter marker list by marker type and by status

# Spectrogram

Clicking on the icon “Spectrogram” will show the spectrogram traces.

Open patient database    Open Trending    Open spectrogram    Add comment    Navigate and Zoom Global time line    Check connection    Change settings

Global time line →

Spectrogram →

Spectrogram Intensity →

Raw EEG trace →

Add marker →

Filter Markerlist

Edit marker

Navigate and Zoom Spectrogram widget

Evaluate marker

Add marker to report

Change montage    Apply filters    Start autoscroll    Select time    Start Zoom mode    Apply default Zoom    Change resolution    Step through EEG    Step through markers

Start Measurement Tool

The spectrogram adds the following functionalities:

- Display of the spectrogram of the channels D-C and C-P.
- Get the estimated frequency in the spectrogram trace
- Adjust the color settings by changing the maximum intensity shown.

## Report

Clicking on “Report” will open the report window.

**Automatically generated content**

**Clear**

**Export to PDF**

**Fill out**

**Copy automatically comment**

**Report**

Reported by: \_\_\_\_\_

Reference period: 2017-01-13 to 2017-02-20

Current period: 2017-02-21 to 2017-03-29

Indication for testing: \_\_\_\_\_

Summary of the findings: \_\_\_\_\_

Diagnostic significance: \_\_\_\_\_

Clinical comments: \_\_\_\_\_

Paste from comment

**Patient Information**

Name: Michael Jones | Treating physician: unselected

Sex: unselected | Seizure Types: \_\_\_\_\_

Date of birth: 1970-01-01 | Implant ID: E 06

Age at study time: 47 | Location: Left

**Recording and report summary**

	Reference period	Current period
Days of recording	39	37
Usage (%)	62%	61%
Electrographic seizures	4	0
Electrographic seizure rate per 24 hours	0.10	0.00
Patient-reported events	0	0

**Seizures over time**

Legend: Weekend data, Weekdays data, No data, Electrographic seizure, Patient-reported events

Reference period: [Start] to [End]

Current period: [Start] to [End]

The report offers the following functionalities:

- Enter additional clinical information including:
  - Indication
  - Summary of the findings
  - Diagnostic significance
  - Clinical comments
- Generate content automatically including:
  - Patient information
  - Statistical information
  - Overview of seizure periods over time
  - Seizure periodicity plots
  - Seizure rate over time
  - Examples of seizures
- Paste the EpiSight comment
- Export to a pdf file

## Keyboard shortcuts

The following table shows a summary of keyboard shortcuts available

Modifier	Key	Function
		<b>EEG Plot area</b>
	Arrow left	Scroll EEG-Plot one page to the left
	Arrow right	Scroll EEG-Plot one page to the right
Shift	Arrow left	Scroll EEG-Plot one second to the left
Shift	Arrow right	Scroll EEG-Plot one second to the right
	Home	Scroll EEG-Plot to the beginning of the recording
	End	Scroll EEG-Plot to the end of the recording
Ctrl	+	Zoom one step into the EEG Area
Ctrl	-	Zoom one step out of the EEG Area
Ctrl	0	Zoom EEG Area to default value
Shift	MouseWheel	Zoom at the cursor position in EEG Area
Ctrl	M	Enter measurement mode
	Esc	Exit measurement mode
		<b>Marker List</b>
	Arrow up	Scroll one row up in the marker list
	Arrow down	Scroll one row down in the marker list
Shift	Arrow up	Expand selection with next marker up
Shift	Arrow down	Expand selection with next marker down
Ctrl	A	Select all marker currently visible in the marker list
Ctrl	C	Copy selected marker to the clipboard
	A	Rate selected markers as "approved"
	R	Rate selected markers as "rejected"
	U	Rate selected markers as "uncertain"
		<b>Trending and Spectrogram</b>
Ctrl	S	Show spectrogram
Ctrl	T	Show trending
		<b>Edit Comment</b>
Ctrl	E	Open the comment dialog
Ctrl	Enter	Close patient comment dialog and save comment modifications
	Esc	Close patient comment dialog and discard comment modifications

<b>Spectrogram and Trending Widget</b>		
Ctrl	Arrow left	Scroll spectrogram/trending plot one page to the left
Ctrl	Arrow right	Scroll spectrogram/trending plot one page to the right
Ctrl + Shift	Arrow left	Scroll spectrogram/trending plot 30 seconds to the left
Ctrl + Shift	Arrow right	Scroll spectrogram/trending plot 30 seconds to the right

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
The screenshot displays the UNEEG EpiSight Analyzer interface. On the left, a 'Patients' table lists patient records with columns for Patient ID, Last name, First name, Birth date, Connection status, and an 'Open' button. The main area shows 'Details for Michael Jones - Connected', including patient information, a list of UNEEG devices, a comment field, and data information such as recording start/end times and data availability. A 'Data overview' section at the bottom shows a timeline of data availability and analysis. Various icons and buttons are annotated with German labels indicating their functions.

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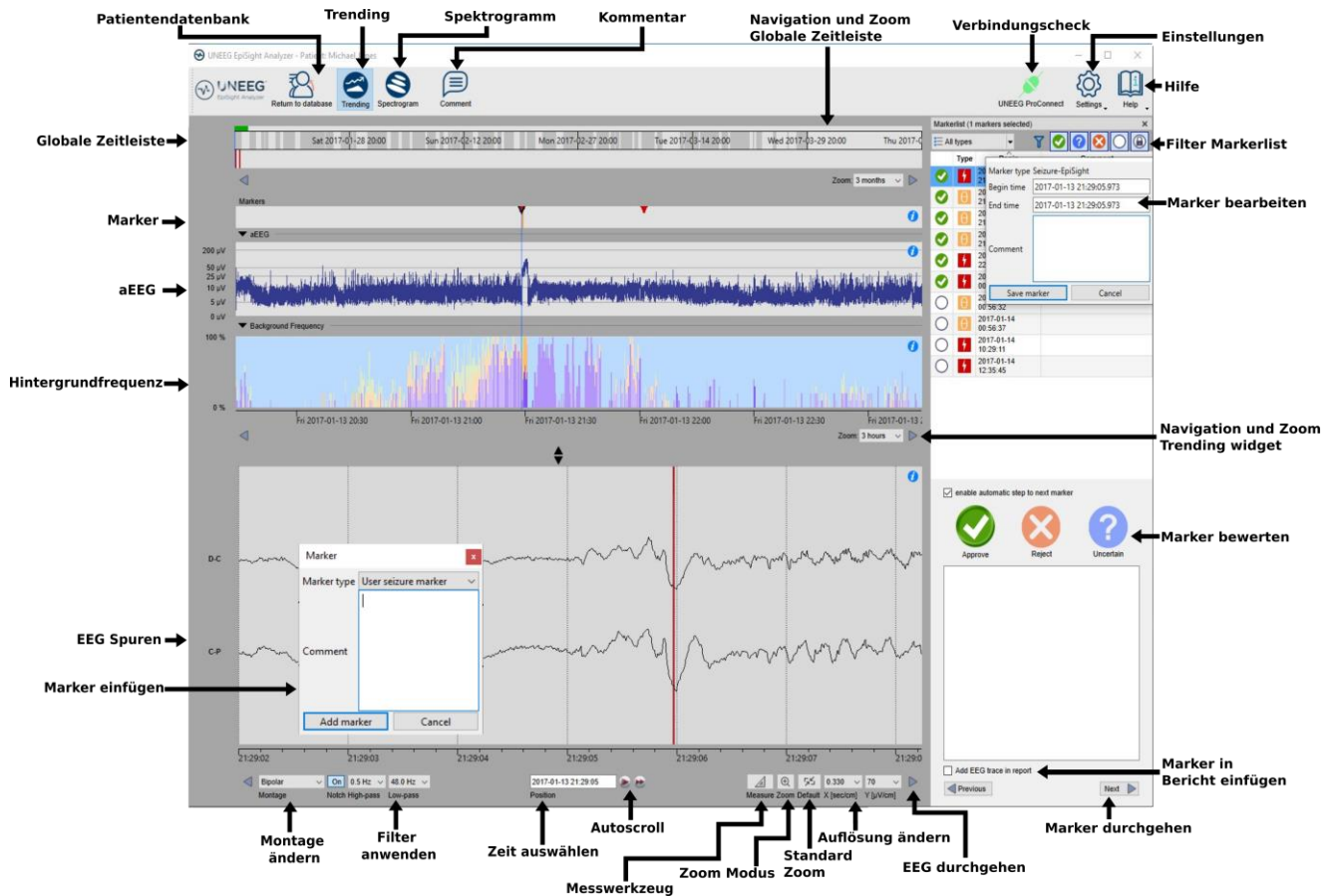The screenshot illustrates the UNEEG EpiSight Analyzer interface, specifically the Spectrogram feature. The interface is divided into several sections:

- Top Menu Bar:** Includes options like Patientendatenbank, Trending, Spektrogramm, Kommentar, Navigation und Zoom Globale Zeitleiste, and Verbindungscheck.
- Global Timeline (Globale Zeitleiste):** Shows a timeline from Saturday, 2017-01-13 20:00 to Thursday, 2017-01-19 21:29:00.
- Spectrogram:** Displays two channels (D-C and C-P) with frequency bands (32.0 Hz, 8.0 Hz, 2.0 Hz, 0.5 Hz) and intensity. A color scale at the bottom indicates intensity levels from 1.4 µV to 14.1 µV.
- EEG Traces (EEG Spuren):** Shows the raw EEG traces for D-C and C-P channels.
- Marker List (Markerlist):** A table on the right side showing markers with columns for Type, Begin, and Comment. It includes buttons for 'Approve', 'Reject', and 'Uncertain'.
- Marker Dialog (Marker einfügen):** A dialog box for adding a marker, showing 'Marker type: User seizure marker' and a 'Comment' field.
- Bottom Controls:** Includes buttons for Montage ändern, Filter anwenden, Zeit auswählen, Autoscroll, Messwerkzeug, Zoom Modus, Standard Zoom, Auflösung ändern, EEG durchgehen, and Marker durchgehen.

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The screenshot shows the UNEEG EpiSight Analyzer interface. At the top, there are navigation icons for 'UNEEG ProConnect', 'Settings', and 'Help'. Below this is a 'Patients' table with columns for Patient ID, Last name, First name, Birth date, Connection, and Open. A search bar is located above the table. To the right of the table is a 'Details for Michael Jones - Connected' panel, which includes 'Patient information' (Patient ID, Last name, First name, Birth date, Sex, Secure types, Treating physician, Implant serial no., Location), a 'List of UNEEG devices', a 'Comment' text area, and 'Data information' (Start of recording, Current end of recording, Available amount of data, New data since last review, Reviewed until). At the bottom of the details panel is a 'Data overview' chart showing data availability and analysis over time.

Labels and arrows pointing to specific features in the interface:

- Buscar paciente**: Points to the search bar.
- Borrar búsqueda**: Points to the 'Clear search' button.
- Seleccionar columnas**: Points to the 'Select...' dropdown menu.
- Añadir nuevo paciente**: Points to the 'Create new patient' button.
- Comprobar conexión**: Points to the 'UNEEG ProConnect' icon.
- Cambiar los ajustes**: Points to the 'Settings' icon.
- Abrir ayuda**: Points to the 'Help' icon.
- Abrir datos**: Points to the 'Open EEG' button.
- Generar informe**: Points to the 'Open Report' button.
- Importar datos**: Points to the 'Import manually' button.
- Editar detalles**: Points to the 'Edit' icon in the patient details panel.
- Añadir comentario**: Points to the 'Comment' text area.
- Leer información**: Points to the 'Data information' section.

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