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UNEEG EpiSight Analyzer light

Version Nummer 2.4

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The screenshot shows the UNEEG EpiSight Analyzer Patient Database interface. The interface is divided into several sections: a top navigation bar, a patient list table, a detailed patient view, and a data overview section. Annotations with arrows point to various features:

- Patienten suchen**: Points to the search bar in the patient list.
- Suche löschen**: Points to the 'Clear search' button.
- Spaltenauswahl**: Points to the 'Select' dropdown menu.
- Neuen Patienten einfügen**: Points to the 'Create new patient' button.
- Verbindungsscheck**: Points to the 'UNEEG ProConnect' status indicator.
- Einstellungen**: Points to the 'Settings' icon.
- Hilfe**: Points to the 'Help' icon.
- Daten öffnen**: Points to the 'Open EEG' button in the detailed view.
- Bericht erstellen**: Points to the 'Open Report' button.
- Daten importieren**: Points to the 'Import manually' button.
- Details bearbeiten**: Points to the 'List of UNEEG devices' section.
- Kommentar schreiben**: Points to the 'Comment' text area.
- Dateninformation**: Points to the 'Data overview' section.

The patient list table contains the following data:

| Patient ID | Last name | First name | Birth date | Connection | Open |
|------------|-----------|------------|------------|-------------|------|
| JM19700101 | Jones | Michael | 1970-01-01 | Connected | |
| JM19900101 | Williams | John | 1990-01-01 | Unconnected | |
| J51985001 | Smith | Jane | 1985-01-01 | Unconnected | |
| test | | | 1970-01-01 | Unconnected | |

The detailed view for Michael Jones shows the following information:

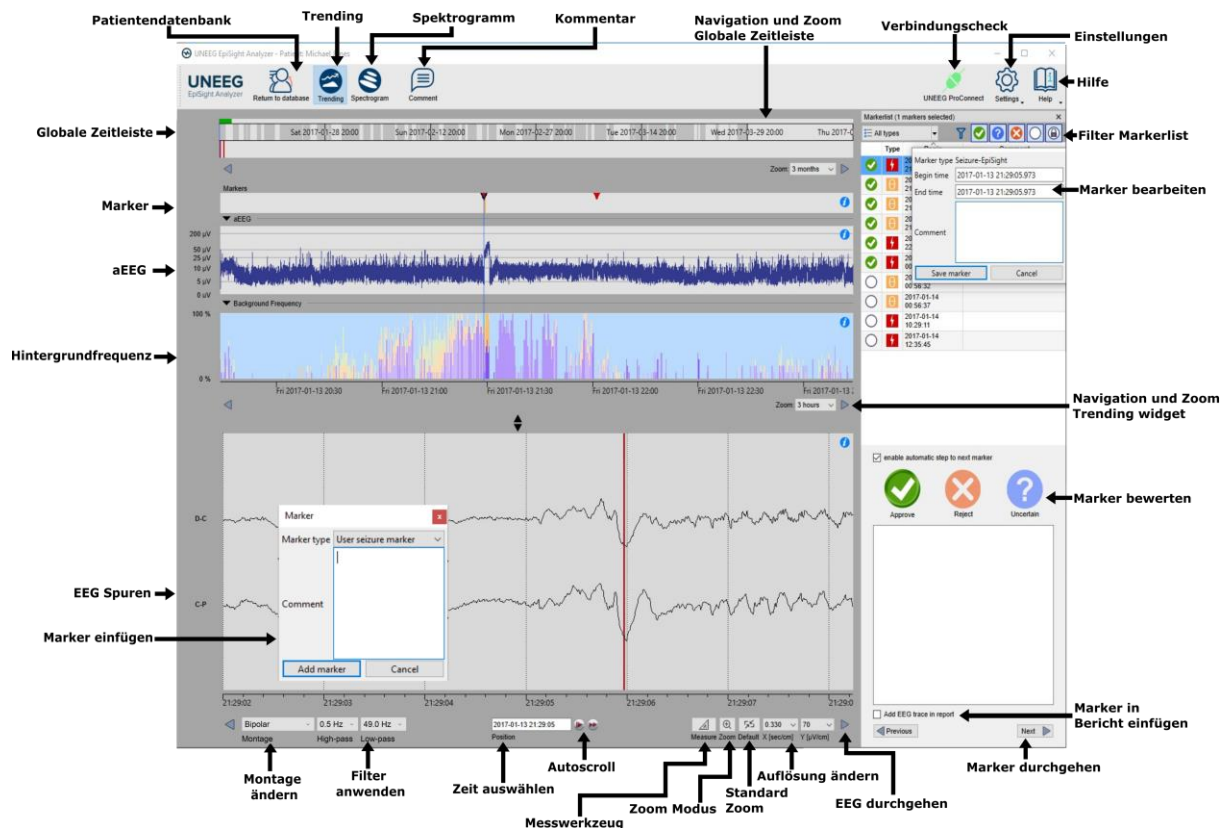
- Patient information**: Patient ID: JM19700101, Last name: Jones, First name: Michael, Birth date: 1970-01-01, Sex: unselected, Seizure type(s): unselected, Treating physician: unselected, Implant serial no.: E 06, Location: Left.
- Data information**: Start of recording: 2017-01-13 20:14:04, Current end of recording: 2017-03-29 22:34:13, Available amount of data: 46 days 16 hours, New data since last review: 46 days 3 hours, Reviewed until: 2017-01-14 12:35.
- Data overview**: A timeline showing data availability (grey) and data analyzed (green) from September 2017 to April 2018.

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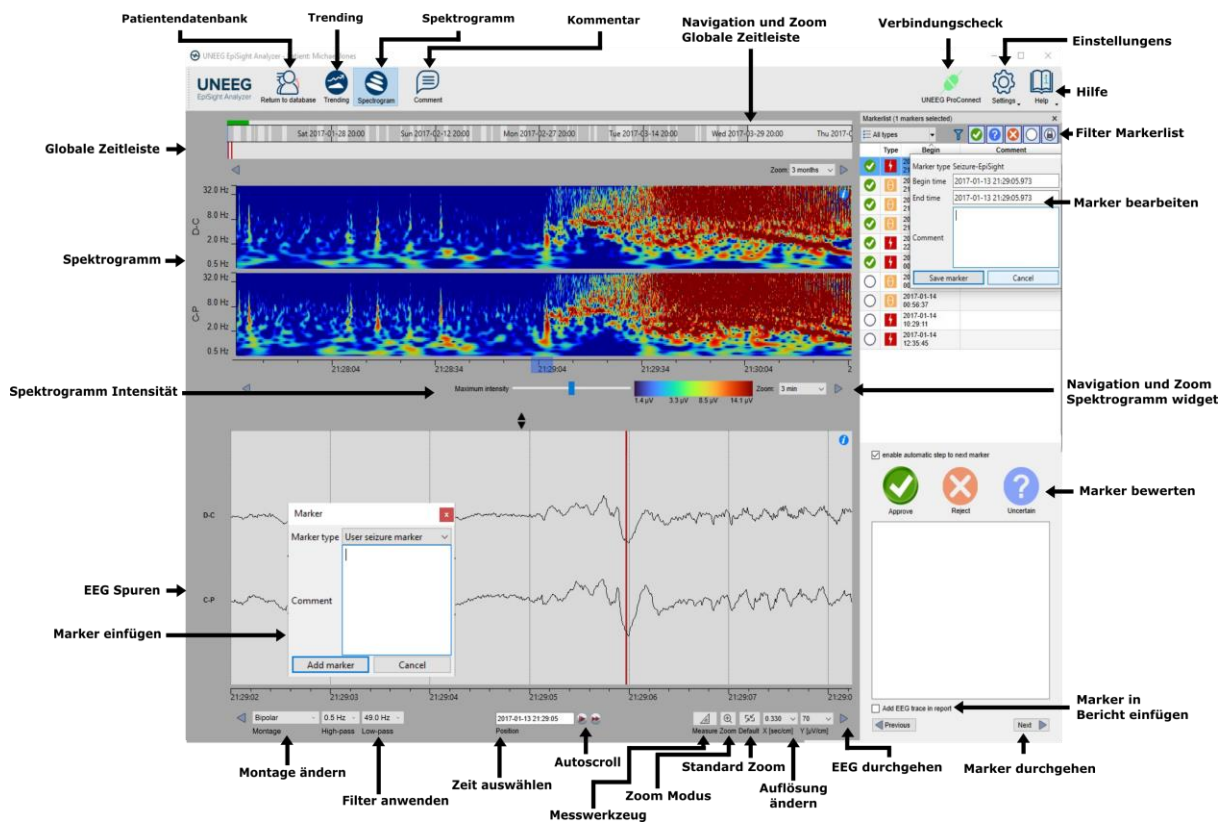
Reference period: [] Current period: []

Reference period start: 2017-01-13 Reference period end: 2017-02-20

Current period start: 2017-02-21 Current period end: 2017-03-29

Indication for testing: []

Summary of the findings: []

Diagnostic significance: []

Clinical comments: []

Patient information

Name: Michael Jones Sex: unselected Date of birth: 1976-01-01 Age at study time: 47

Treating physician: unselected Seizure Types: [] Implant ID: E 06 Location: Left

Recording and report summary

| | Reference period | Current period |
|--|------------------|----------------|
| Days of recording | 39 | 37 |
| Usage (%) | 62% | 61% |
| Electrographic seizures | 4 | 0 |
| Electrographic seizure rate per 24 hours | 0.10 | 0.00 |
| Patient-reported events | 0 | 0 |

Seizures over time

Time of day: 0 to 24

Legend: Weekend data, Weekdays data, No data, Electrographic seizure, Patient-reported events

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